

tweak
ENHANCE™
Rechargeable OTC Hearing Aid

THIS QUICK START GUIDE WILL HELP YOU SET UP ENHANCE OTC HEARING AID. READ THE USER MANUAL INCLUDED WITH THIS OTC HEARING AID FOR COMPLETE INFORMATION.

charging information

- 1 Plug the type-C USB connector into the charging base.
 - 2 Plug the opposite end of the cable into adapter block then connect with power source.
 - 3 The charging base is marked with blue and red circles. Place amplifier in the corresponding port. It will shut off automatically when placed in the charger.
- BLUE: Left RED: Right
- The rocker switch will glow blue while charging. When fully charged, the rocker switch will glow green.

Fully charge Tweak Enhance before proceeding to setup.



connecting tube and dome/tip

- 1 Choose either a left or right sound tube.
- 2 Attach tube connector end to sound post on amplifier. Twist tube clockwise until snug.
- 3 Choose dome/tip to use with sound tube.
- 4 Align tip with end of sound tube and push tip straight on.

wearing tweak enhance otc

- 1 Place Tweak Enhance over your ear. The curve of the sound tube should rest on top of your ear comfortably, even with glasses.
- 2 Grab the loose end of the tubing and gently insert the tip into the ear canal.
- 3 Position the retaining lock in the bowl of the ear.
- 4 When properly inserted, the tubing will lay close to the head around the front of the ear and into the ear canal. Check this by moving your finger down the tubing, into the ear.

download tweak enhance otc app




- 1 Using your smartphone, search for Tweak Enhance OTC app in the App Store or Google Play store.
- 2 Download Tweak Enhance OTC app to your smartphone.

Refer to the APP GUIDE inside the user manual to set up Tweak Enhance OTC hearing aids using the Tweak Enhance OTC app.

using tweak enhance otc

Turning ON and OFF

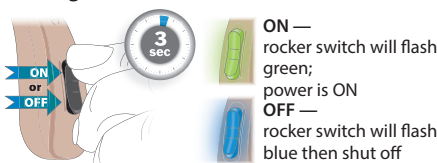

To turn ON or OFF, press and hold the **upper portion** of the rocker switch until you hear a beep (approx. 3 seconds) then release. While you are holding the rocker switch, it will turn green. The green light will turn off once the amplifier is ON.

To turn OFF, press and hold the **upper portion** of the rocker switch until you hear a beep (approx. 3 seconds) then release. While you are holding the rocker switch, it will turn blue. The blue light will turn off once the amplifier is OFF.


Memory Switch

When the hearing instrument is turned ON, it will revert to the last program and volume setting.

Program	Suitable for	Indicator
1	Conversation	1 Beep
2	Restaurant	2 Beeps
3	Traffic	3 Beeps
4	Outdoor	4 Beeps

Volume Control

The rocker switch is used as a volume control. Pressing the upper portion of the rocker switch increases the volume, while the lower portion reduces the volume.

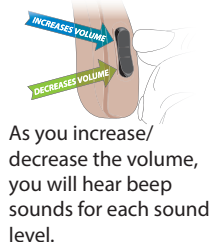

Level 1	Level 2	Level 3	Level 4 MAX Volume
Beep	Beep	Beep	2 Beeps

Level 1	Level 2	Level 3	Level 4 MIN Volume
Beep	Beep	Beep	2 Beeps

To switch programs, press and hold the **lower portion** of the rocker switch for two seconds then release. Do the same to switch to the other programs.

