



### connecting tube and tip





Choose either a left or

right sound tube.



Align the sound tube connector end with the sound post on the hearing aid, then push the tube straight onto the sound post.









Align the dome use with sound tube tip with the end of . ound tube and push



## inserting a battery



 Lift the notch at the edge of the battery compartment and swivel open the door of the battery compartment.



Remove the browncolored tab on a new battery (size 312 zinc-air). Wait one minute for the battery to activate.



3 Notice the edge of the battery door to assist you in finding the proper orientation. Insert the battery with the positive "+" side facing up.



tip straight on.

4 Gently close the battery door. It should close easily; do not force the door closed.

#### vearing tweak



1 Place Tweak over your ear. The curve of the sound tube should rest on top of your ear comfortably, even with glasses.



2 Grab the loose end of the tubing and gently insert the tip into the ear canal.



**3** Position the retaining lock in the bowl of the ear. When properly inserted the tubing will lay close to the head around the front of the ear and into the ear canal. Check this by moving your finger down the tubing, into the ear.

# selecting your amplification

With a fresh battery in your Tweak and wearing the hearing aid, you are ready to choose an amplification profile.

1 Start out in a quiet environment. The first time you turn Tweak on, it will be at the lowest amplification profile. How does it sound?

2 To try the next level of amplification, press and release the upper portion of the rocker





switch. You will hear two beeps, indicating you are now in Amplification Profile 2. What you are doing is sampling the available acoustic settings for quiet environments. There are six different settings for you to try. Choose the one

that sounds best to you—most natural, most helpful volume level.



Continue to press and release either the upper or lower portion of the rocker witch to try out

4 Once vou've made your decision, press and HOLD the rocker switch for six seconds, until you hear a long



To lock in your preferred amplification profile, hold the rocker switch (upper or lower) for 6 seconds until you hear a

long beep.

beep. This procedure locks in your chosen amplification profile. Three programs are now available: Quiet, Noise, and Telecoil

Congratulations, your set up is now complete! Tweak Focus+T remembers your preferred amplification profile from now on.

## amplification profiles er switch is earing aid



the six different



d the progr econd then







tweakhearing.com | 1-888-815-3240 | info@tweakhearing.com