

# Quick Start Guide

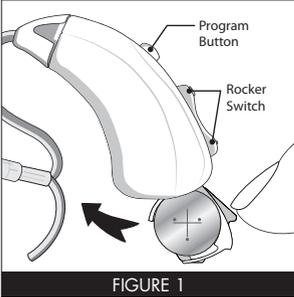


FIGURE 1

## 1. Prepare for use

Attach either a left or right tube.

Insert battery and close the battery door. Your Tweak is now on. (Figure 1)

## 2. Place Tweak over your ear.

Position the tip in the ear canal and the retaining lock in the bowl of your ear. (Figure 2)

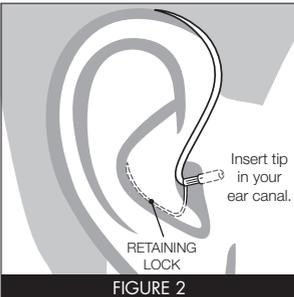


FIGURE 2

## 3. Try out different amplification settings

Start out in a relatively quiet environment. When you first turn Tweak on, it will be at the lowest level of amplification. How does it sound?

There are three different amplification settings for you to try. Press and release the rocker switch. (Figure 3)

You will hear two (2) beeps, indicating you are now at the next setting, and so on, up to three levels of amplification. Use the rocker switch to go up or down to try out the different settings.

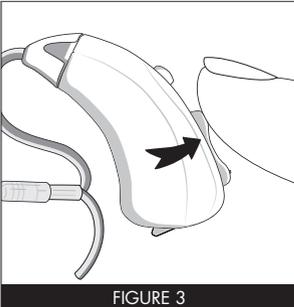


FIGURE 3



## 4. Lock in your preferred setting

Once you decide which amplification level you like best, **press and HOLD the rocker switch for six (6) seconds, until you hear a long beep.** (Figure 4)

Congratulations! From this point forward Tweak remembers the amplification profile you just set.

Now you can switch between three different listening modes—Quiet, Noise, and Telecoil.

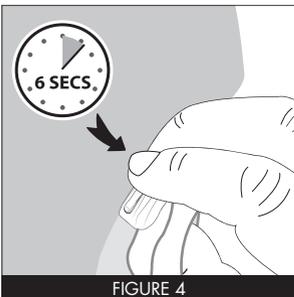


FIGURE 4

To switch between the three listening modes, **press and release the program button.** Once you release it you will hear a confirmation: a single beep for Quiet, “Shhh” sound for Noise, and a dial tone for Telecoil. *In all programs, you can still adjust the volume up or down by pressing the upper or lower rocker switch.*

**Important:** To extend battery life, turn off your Tweak by opening the battery door whenever the device is not in use.

tweak

www.TweakHearing.com | 1-888-815-3240 | info@TweakHearing.com