


Quick Start Guide

For complete instructions, refer to User Instruction Manual.

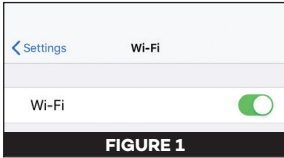

1. Enable Wi-Fi on your iPhone

Go to iPhone **Settings**, select **Wi-Fi**. On the next screen, move slider to the right to turn on Wi-Fi. Close Settings. (FIGURE 1)



2. Enable Bluetooth on your iPhone

Go to **Settings**, tap **Bluetooth**. Move slider to the right to turn on Bluetooth. Close Settings. (FIGURE 2)




3. Locate the TweakConnect Application

The application (often abbreviated and referred to as an 'app') can be downloaded to your iPhone from The App Store, which can typically be found on the main screen of your iPhone. (FIGURE 3)




4. Download and install the app to your iPhone

In The App Store, locate the TweakConnect app. Select **GET** to begin downloading the app. (FIGURE 4)

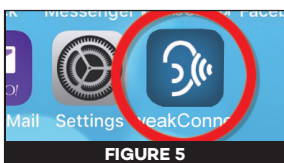

5. Launch the TweakConnect App

Start the app by tapping it on your iPhone screen, or selecting **OPEN** if the app installation process was left at that stage. (FIGURE 5)

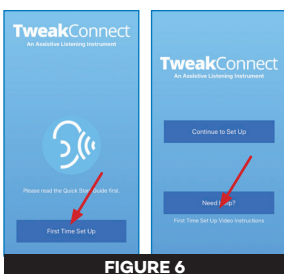

6. First Time Set Up

Select **First Time Set Up**. On the second screen, select **Need Help?** to watch the videos. For first time users, play the **Getting Started** and **Using the App** videos, which will provide a tutorial for how to use the app. (FIGURE 6)

7. Proceed to set up

Screen prompts will guide you through each of the steps to personalize your sound on the TweakConnect device.

! IMPORTANT

Do not install the battery or close the battery door until prompted to do so by the app.