

TWEAK Focus+T Quick Start Guide

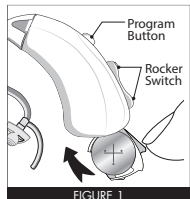


FIGURE 1

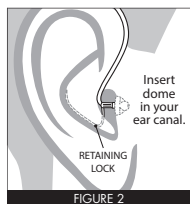


FIGURE 2

1. Prepare for use. Attach either a left or right tube. Insert battery and close the battery door. Your Tweak is now on. (Figure 1)

2. Place Tweak over your ear. Position the tip in the ear canal and the retaining lock in the bowl of your ear. (Figure 2)


3. Try out different amplification settings

Start out in a quiet environment. When you first turn Tweak on, it will be at the lowest level of amplification. How does it sound?

There are four different amplification settings for you to try. Press and release the upper rocker switch.

You will hear two (2) beeps, indicating you are now at the next setting, and so on, up to four levels of amplification. (Figure 3) Use the rocker switch to go up or down to try out the different settings.

Continue sampling the settings until you decide which one you like best.

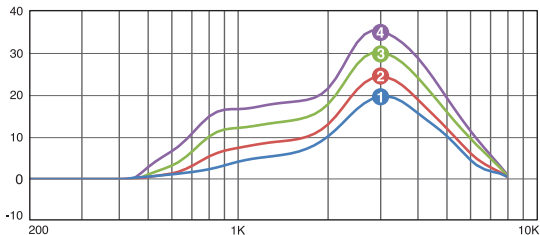

AMPLIFICATION SETTINGS

OMNI-DIRECTIONAL | DIRECTIONAL | TELECOIL

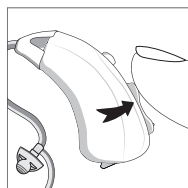


FIGURE 3



FIGURE 4

4. Lock in your preferred setting. Press and HOLD the rocker switch for six (6) seconds, until you hear a long beep. (Figure 4)

Congratulations! From this point forward Tweak remembers the amplification profile you just set. Now you can switch between three different listening modes – Quiet, Noise, and Telecoil.

To switch between the three listening modes, press and release the program button. Once you release it you will hear a confirmation: a single beep for Quiet, “Shhh” for Noise, and a dial tone for Telecoil.

To adjust the volume in any program, tap the upper or lower rocker switch.

If you get feedback – “whistling” sound – try decreasing the volume first. If feedback continues, consider using one of the larger vented or closed domes. You can also change to the thick sound tube with earhook and a closed dome.

RESET



Instrument may be reset by following these steps:

- 1) With a battery inserted, open the battery door.
- 2) Press and hold the rocker switch.
- 3) While still pressing and holding the rocker switch, close the battery door.
- 4) Wait at least 14 seconds, then release the rocker switch.
- 5) Open the battery door.
- 6) When you close the battery door again, Tweak is reset.




You may now proceed to select a new Amplification Setting.

Important: To extend battery life, turn your Tweak off by opening the battery door whenever the device is not in use.

OMNI-DIRECTIONAL | DIRECTIONAL | TELECOIL

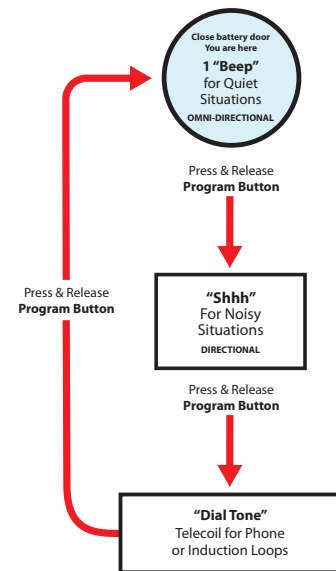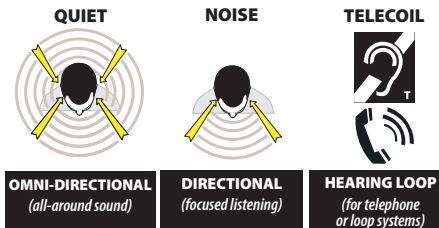

OMNI-DIRECTIONAL | DIRECTIONAL | TELECOIL

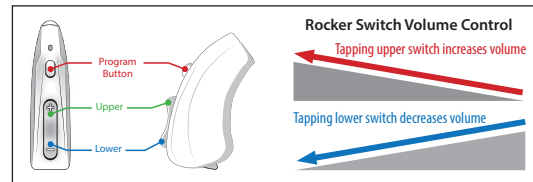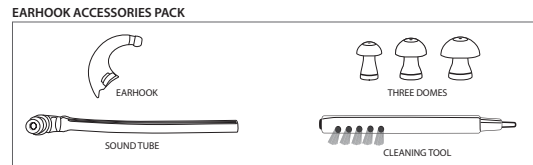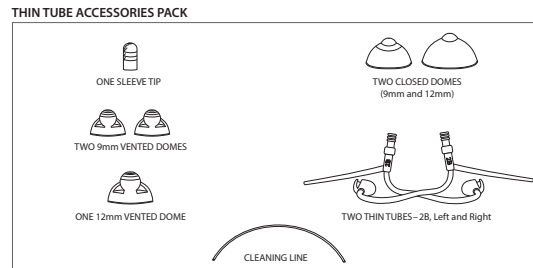

tweak

www.TweakHearing.com | 1-888-815-3240 | info@TweakHearing.com

worldwide patents: www.eartech.com/patents

TWF+T QSG 0119 Rev B